

## 6 Tips for Using Alpha-Stim on Animals

### 1. If pet is restless, do 10 minutes of CES first

Apply Earclips for a short treatment first. Then, follow with probe treatment. Complete with final 10 minutes of CES treatment.

### 2. Be sure to use enough conducting solution

When doing a probe treatment, re-apply 1-2 drops of solution onto the probe electrode pads (PEPs) for every minute of use.

### 3. Wet the fur of the animal

A water spray bottle can be used to wet the fur. This aids in the probe treatment making connection through the fur barrier. Fluff fur backwards as spraying to allow the moisture to dampen the fur at the skin level.

### 4. Hold probes at a correct angle to the body

The open spot in the center of each probe pad allows the probe to connect to the skin of the animal; it must be perpendicular to the skin. Gently circle the probes through the fur to establish connection with skin, then press gently to indent slightly.

### 5. Move one probe at a time, and move it quickly

This will keep the connection going and take less time to complete the therapy. When both probes are removed from contact at the same time, the connection is lost. Re-establishing connection can take a couple seconds

### 6. When doing the probe treatment, use the infinity setting for time

Watch the clock count upward to track time. This avoids the 10 second BEEP sound that you hear when using the probe timer setting and allows for smoother movement between points without the reset being required.

